



In House Test List & Stability Requirements

Test ID	Test Name	CPT Code	Storage	Specimen	Room Temperature	Refrigerated	Frozen
24UCA	24 Hour Urine Calcium	82340	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UCL	24 Hour Urine Chloride	82438	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UCRE	24 Hour Urine Creatinine	82570	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UK	24 Hour Urine Potassium	84132	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UMALB	24 Hour Microalbumin	82043	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UNA	24 Hour Urine Sodium	84300	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
24UPRO	24 Hour Urine Protein	84156	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
ABID	Antibody Identification	86870	Refrigerate	Lavender (1)	24 hours	7 days	UNACCEPTABLE
ABORH	ABO/RH	86900,86901	Refrigerate	Lavender (1)	8 hours	72 hours	-----
ABSC	Antibody Screen	86850	Refrigerate	Lavender (1)	8 hours	72 hours	-----
ACUL	Culture, Anaerobic	87075	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
ALB	Albumin	82040	Refrigerate	SST (1)	8 hours	48 hours	-----
ALKP	Alkaline Phosphatase	84075	Refrigerate	SST (1)	8 hours	48 hours	-----
ALT	ALT (SGPT)	84460	Refrigerate	SST (1)	8 hours	48 hours	-----
AMPC	Amphetamine Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	UNACCEPTABLE
AMYL	Amylase	82150	Refrigerate	SST (1)	8 hours	48 hours	-----
AMYU	Random Urine Amylase	82150	Refrigerate	Urine Cup (1)	7 days	7 days	28 days
APTT	Activated Partial Thromboplastin Time	85730	Refrigerate	Blue Top (1)	4 hours	4 hours	2 weeks
AST	AST (SGOT)	84450	Refrigerate	SST (1)	8 hours	48 hours	-----
ASTR	Rapid Strep Group A	87081	Refrigerate	Swab (1)	8 hours	72 hours	UNACCEPTABLE
BARC	Barbiturates Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
BCUL	Culture, Blood	87040	Room Temperature	BC Bottle (1)	24 hours	UNACCEPTABLE	UNACCEPTABLE
BENC	Benzo Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
BMP	Basic Metabolic Panel	80048	Refrigerate	SST (1)	8 hours	48 hours	-----
BNP	BNP	83880	Refrigerate	Lavender (1)	8 hours	24 hours	9 months
BUN	BUN	84520	Refrigerate	SST (1)	8 hours	48 hours	-----
C125	CA 125	86304	Refrigerate	SST (1)	8 hours	48 hours	-----
C199	CA 19-9	86301	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CA	Calcium	82310	Refrigerate	SST (1)	8 hours	48 hours	-----
CA15	CA 15-3	86300	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CBC	Complete Blood Count with Auto Diff	85025	Refrigerate	Lavender (1)	24 hours	48 hours	-----
CBCMD	Complete Blood Count with Manual Diff	85025	Refrigerate	Lavender (1)	24 hours	48 hours	-----
CBCND	CBC with No Diff	85027	Refrigerate	Lavender (1)	24 hours	48 hours	-----
CCFL	Fluid Cell Count	89051	Room Temperature	Lavender (1)	24 hours or ASAP	-----	-----
CDT	C-Diff Toxin	87493	Refrigerate	Sterile Container (1)	24 hours	4 days	DO NOT FREEZE
CEA	CEA	82378	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CHOL	Cholesterol	82465	Refrigerate	SST (1)	8 hours	48 hours	-----
CKMB	CKMB	82553	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CL	Chloride	82435	Refrigerate	SST (1)	8 hours	48 hours	-----
CMP	Comprehensive Metabolic Panel	80053	Refrigerate	SST (1)	8 hours	48 hours	-----
CO2	CO2	82374	Refrigerate	SST (1)	8 hours	48 hours	-----
COCC	Cocaine Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
CORA	Cortisol AM	82533	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED

CORP	Cortisol PM	82533	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CORT	Cortisol	82533	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
CPK	Creatine Kinase	82550	Refrigerate	SST (1)	4 hours	8-12 hours	2-3 days
CREA	Creatinine	82565	Refrigerate	SST (1)	8 hours	48 hours	-----
CRECLR	Creatinine Clearance	82575	Refrigerate	Urine Cup (1)	Do not leave at RT	48 hours	2 months
CRP	C-Reactive Protien	86140	Refrigerate	SST (1)	8 hours	72 hours	-----
CRPH	C-Reactive Protein (High Sensitivity)	86141	Refrigerate	SST (1)	8 hours	72 hours	-----
CRYF	Fluid Crystals	89060	Room Temperature	Lavender (1)	24 hours or ASAP	-----	-----
CTGCRNA	CT/GC RNA Probe	87591,87491	Room Temperature	Probe (1)	2 weeks *urine must be transferred within 1 hr*	30 days	12 months
CXHG	Carboxyhemoglobin	82375	Refrigerate	Lavender (1)	5 days	5 days	UNDETERMINED
DAP5	Drug Abuse Panel 5	80307	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
DAP7	Drug Abuse Panel 6	80307	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
DAP8	Drug Abuse Panel 8	80307	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
DAP9	Drug Abuse Panel 9	80307	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
DBIL	Bilirubin, Direct	82248	Refrigerate	SST (1)	8 hours	48 hours	-----
DC	Direct Coombs	83880	Room Temperature	Lavender (1)	8 hours	72 hours	-----
DDIM	D-Dimer	85378	Frozen	Blue Top (1)	4 hours	-----	90 days
DIG	Digoxin	80162	Refrigerate	Red Top (1)	8 hours	48 hours	-----
ECUL	Environmental Cutlure	87071	Refrigerate	Swab (1)	24 hours	48 hours	-----
EMDS	Drug Screen- Preemployment	80307	Refrigerate	Urine Cup (1)	8 hours	7 days	UNDETERMINED
ESR	Erythrocyte Sedimentation Rate	85651	Refrigerate	Lavender (1)	4 hours	24 hours	-----
ESTRA	Estradiol	82670	Refrigerate	SST (1)	8 hours	7 days	UNDETERMINED
FAMY	Fluid Amylase	82150	Room Temperature	Red Top (1)	7 days	28 days	28 days
FERR	Ferritin	83540	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
FLDH	Fluid LDH	83615	Room Temperature	Red Top (1)	7 days	4 days	42 days
FLUAB	Influenza Virus A and B	86710, 87804	Room Temperature	Swab (1)	8 hours	-----	-----
FOLT	Folate	82746	Refrigerate	SST (1)	Refrigerate ASAP	48 hours	6 months
FPRO	Fluid Protein	84157	Room Temperature	Red Top (1)	7 days	28 days	28 days
FSH	Follicle-Stimulating Hormone	83001	Refrigerate	SST (1)	8 hours	48 hours	3 months
FSLH	FSH & LH	83001,83002	Refrigerate	SST (1)	8 hours	48 hours	3 months
FT3	Free T3	84481	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
FT4	Free T4	84439	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
GCUL	Culture, Genital	87070	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
GENT	Gentamicin	80170	Refrigerate	SST (1)	8 hours	48 hours	-----
GGT	GGT	82977	Refrigerate	SST (1)	8 hours	48 hours	-----
GLFL	Fluid Glucose	82945	Room Temperature	Red Top (1)	7 days	28 days	28 days
GLU	Glucose	82947	Refrigerate	SST (1)	8 hours	48 hours	-----
GPP2H	Glucose PP/2HR	82947	Refrigerate	SST (1)	8 hours	48 hours	-----
GRBC	Culture, Group B Strep	87081	Refrigerate	Swab (1)	24 hours	48 hours	-----
GRBST	Group B Strep	87653	Refrigerate	Swab (1)	-----	24 hours	-----
GRST	Gram Stain	87205	Refrigerate	Sterile Container (1)	24 hours	-----	-----
GTT1H	1 Hour GTT (OB Screen)	82950	Refrigerate	SST (1)	8 hours	48 hours	-----
GTT2	Glucose Tolerance Test 2 Hour	82947,82950, 82950	Refrigerate	SST (1)	8 hours	48 hours	-----
GTT3	Glucose Tolerance Test 3 Hour	82947,82950, 82950	Refrigerate	SST (1)	8 hours	48 hours	-----
GTT4	Glucose Tolerance Test 4 Hour	82947,82950,82950,82952	Refrigerate	SST (1)	8 hours	48 hours	-----
GTT5	Glucose Tolerance Test 5 Hour	82947,82950,82950,82953	Refrigerate	SST (1)	8 hours	48 hours	-----
HAGM	Hepatitis A IgM	86709	Refrigerate	SST (1)	Immediately	7 days	180 days
HBA1C	Hemoglobin A1C	83036	Refrigerate	Lavender (1)	8 hours	7 days	3 months
HBCM	Hepatitis B Core Antibody IgM	86705	Refrigerate	SST (1)	Immediately	7 days	UNDETERMINED

HBSA	Hepatitis B Surface Antigen	87340	Refrigerate	SST (1)	Immediately	48 hours	UNDETERMINED
HCVAB	Hepatitis C Antibody	86803	Refrigerate	SST (1)	Immediately	48 hours	-----
HDL	HDL	83718	Refrigerate	SST (1)	8 hours	7 days	30 days
HEPP1	Hepatitis I Panel	86709,86705,87340,86803	Refrigerate	SST (1)	8 hours	48 hours	-----
HEPPNL	Hepatic Function Panel	8,82247,82040,84155,84450,84075,8	Refrigerate	SST (1)	8 hours	48 hours	-----
HH	Hemoglobin and Hematocrit	85014,85018	Refrigerate	Lavender (1)	24 hours	48 hours	-----
HIV	HIV Antibody	86701	Refrigerate	SST (1)	Immediately	48 hours	365 days
HOMOP	Homocysteine- Plasma	83090	Refrigerate	Lavender (1)	4 days	4 days	-----
HOMOS	Homocysteine- Serum	83090	Refrigerate	SST (1)	4 days	4 days	-----
HPV	Human Papillomavirus	87624	Room Temperature	Probe (1)	2 weeks	30 days	12 months
HPVSP	HPV Subtype Profile	87625	Room Temperature	Probe (1)	2 weeks	30 days	12 months
HYPOT	Hypothyroid Panel	84479,84436,84443	Refrigerate	SST (1)	8 hours	24 hours	-----
INSU	Insulin	83525	Refrigerate	SST (1)	8 hours	24 hours	UNDETERMINED
IRNPNL	Iron Panel	83540,84466	Refrigerate	SST (1)	-----	48 hours	-----
IRON	Iron	83540	Refrigerate	SST (1)	Recentrifuge if added on after 24 hours	48 hours	-----
K	Potassium	84132	Refrigerate	SST (1)	8 hours	48 hours	-----
KLEB	Kleihauer Betke Stain	80177	Room Temperature	Lavender (1)	72 hours	72 hours	-----
LAAC	Lactic Acid	83605	Refrigerate	Gray Top (1)	8 hours	14 days	14 days
LDH	LDH	83615	Refrigerate	SST (1)	2 hours or ASAP	-----	-----
LDLD	LDL-Direct	83721	Refrigerate	SST (1)	8 hours	5 days	-----
LH	Luteinizing Hormone	83002	Refrigerate	SST (1)	8 hours	48 hours	6 months
LI	Lithium	80178	Refrigerate	SST (1)	8 hours	48 hours	-----
LIPID	Lipid Panel	83718,82465,84478	Refrigerate	SST (1)	8 hours	48 hours	-----
LPAS	Lipase	83690	Refrigerate	SST (1)	4 hours	48 hours	-----
LYTES	Electrolyte Panel	80051	Refrigerate	SST (1)	8 hours	48 hours	-----
MACR	Microalbumin/Creatinine, Random	82043,82570	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
MALR	Microalbumin Random	82043	Refrigerate	Urine Cup (1)	8 hours	72 hours	-----
MDIFF	Manual Differential	85007	Room Temperature	Slide (1)	24 hours	24 hours	-----
METC	Methadone Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	-----
MG	Magnesium	83735	Refrigerate	SST (1)	8 hours	48 hours	-----
MONS	Mono Spot Screen	86308	Refrigerate	SST (1)	24 hours	48 hours	-----
MRSA	MRSA by PCR	87641	Room Temperature	Swab (1)	24 hours	7 days	-----
MYO	Myoglobin	83874	Refrigerate	SST (1)	8 hours	24 hours	UNDETERMINED
NA	Sodium	84295	Refrigerate	SST (1)	8 hours	48 hours	-----
NH3	Ammonia	82140	Transport on Ice	Green Top (1)	Immediately put on Ice, needs to be ran within 30 minutes		
NIDT	NIDA/DAP5 DOT	80300	Refrigerate	Urine Cup (1)	8 hours	7 days	-----
OCBL	Occult Blood	82272	Refrigerate	FOB Card (1)	8 hours	48 hours	-----
PALB	Prealbumin	84134	Refrigerate	SST (1)	8 hours	48 hours	-----
PBAR	Phenobarbital	80184	Refrigerate	Red Top (1)	8 hours	48 hours	-----
PCPC	Phencyclidine Confirmation	80102	Refrigerate	Urine Cup (1)	8 hours	7 days	-----
PHEN	Phenytoin (Dilantin)	80185	Refrigerate	Red Top (1)	8 hours	48 hours	-----
PHOS	Phosphorus	84100	Refrigerate	SST (1)	8 hours	48 hours	-----
PREG	Serum Pregnancy	84703	Refrigerate	SST (1)	8 hours	48 hours	-----
PRGS	Progesterone	84144	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
PRLA	Prolactin	84146	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
PROCAL	Procalcitonin	84145	Frozen	SST (1)	2 days	48 hours	3 months
PSA	Prostatic Specific Antigen, Total	84153	Refrigerate	SST (1)	Immediately spin and separate	24 hours	UNDETERMINED

PTAP	PTINR and APTT	85610,85730	Room Temperature	Blue Top (1)	4 hours unless separated and frozen	-----	-----
PTHINT	Intact PTH	83970	Refrigerate	SST (1)	4 hours	8 hours	6 months
PTINR	Prothrombin Time with INR	85610	Room Temperature	Blue Top (1)	24 hours	-----	-----
PXEC	Propoxyphene Confirmation	80102	Refrigerate	Urine Cup (1)	5 days	7 days	30 days
QHCG	Beta hCG, Quantitative	84702	Refrigerate	SST (1)	8 hours	48 hours	6 months
RENAL	Renal Function Panel	84520,82310,82435,82374,82565,82947,84132,84295,84100	Refrigerate	SST (1)	8 hours	48 hours	-----
RETI	Retic Count	85045	Refrigerate	Lavender (1)	24 hours	72 hours	-----
RGLU	Random Urine Glucose	82947	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
ROTA	Rotavirus	87425	Frozen	Sterile Container (1)	ASAP	72 hours	-----
RPR	RPR (VDRL)	86592	Refrigerate	SST (1)	24 hours	48 hours	-----
RUB	Rubella	86762	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
RUCA	Random Urine Calcium	82340	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUCL	Random Urine Chloride	82436	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUCR	Random Urine Creatinine	82570	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUK	Random Urine Potassium	84133	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUMG	Random Urine Magnesium	83735	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUNA	Random Urine Sodium	84300	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUPHO	Random Urine Phosphorous	84105	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RURI	Random Urine Uric Acid	84560	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
RUTP	Random Urine Protein	82042	Refrigerate	Urine Cup (1)	2 hours or ASAP	48 hours	-----
SALC	Blood Alcohol	82055	Refrigerate	Green Top (1)	Do not remove cap, must be ran within 8 - 10 hours		
SALI	Salicylate	80196	Refrigerate	SST (1)	7 days	7 days	7 days
SHB	Sex Hormone Binding Globulin	84270	Refrigerate	SST (1)	8 hours	7 days	2 months
STCL	Culture, Stool	87045	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
T3U	T3 Uptake	84479	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
TBIL	Bilirubin, Total	82247	Refrigerate	SST (1)	8 hours	48 hours	-----
TCUL	Culture, Throat	87070	Refrigerate	Swab (1)	24 hours	48 hours	-----
TEGR	Tegretol(Carbamazepine)	80156	Refrigerate	Red Top (1)	8 hours	48 hours	-----
TEST	Testosterone	84403	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
TFTS	Testosterone, Free & Total	84270, 84403	Refrigerate	SST (1)	8 hours	48 hours	2 months
THCC	Cannabinoid Confirmation	80102	Refrigerate	Urine Cup (1)	72 hours	72 hours	-----
THEO	Theophylline	80198	Refrigerate	Red Top (1)	8 hours	48 hours	-----
THPX	Thyroid Peroxidase Autoantibody	86376	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
TOT4	Total T4	84436	Refrigerate	SST (1)	8 hours	24 hours	UNDETERMINED
TP	Total Protein	84155	Refrigerate	SST (1)	Spin and separate within 2 hours	48 hours	-----
TRFR	Transferrin	84466	Refrigerate	SST (1)	8 hours	48 hours	-----
TRICH	Trichomonas	87798	Room Temperature	Probe (1)	2 weeks	30 days	-----
TRIG	Triglyceride	84478	Refrigerate	SST (1)	8 hours	48 hours	-----
TRO1	Troponin I	84484	Refrigerate	SST (1)	Immediately spin and separate	24 hours	1 month (centrifuge for 10 minutes before analysis)
TSFT	TSH & Free T4	84439,84443	Refrigerate	SST (1)	8 hours	48 hours	UNDETERMINED
TSH	TSH	84443	Refrigerate	SST (1)	18 hours	7 days	90 days
UA	Urinalysis, Complete	81001	Refrigerate	Urine Cup (1)	12 hours	48 hours	-----
UACNS	Urinalysis with CNS if Indicated	81001	Refrigerate	Urine Cup (1)	12 hours	48 hours	-----
UALC	Urine Alcohol	82055	Refrigerate	Urine Cup (1)	ASAP	-----	-----

UAWM	Urinalysis with Microscopic	81001	Refrigerate	Urine Cup (1)	12 hours	48 hours	-----
UCUL	Culture, Urine	87086	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
UMIC	Urinalysis, Microscopic	81015	Refrigerate	Urine Cup (1)	12 hours	48 hours	-----
UOPQ	Opiate Confirmation	80102	Refrigerate	Urine Cup (1)	72 hours	72 hours	-----
UPCR	Protein/Creatinine, Urine	82043,82570	Refrigerate	Urine Cup (1)	8 hours	48 hours	-----
URFL	Fluid Uric Acid	84560	Room Temperature	Red Top (1)	8 hours	48 hours	-----
URIC	Uric Acid	84550	Refrigerate	SST (1)	8 hours	48 hours	-----
URPG	Urine Pregnancy	81025	Refrigerate	Urine Cup (1)	-----	48 hours	UNDETERMINED
UUN	Random Urine Urea Nitrogen	84540	Refrigerate	Urine Cup (1)	8 hours	48 hours	-----
VAL	Valproic Acid(Depakote)	80164	Refrigerate	Red Top (1)	8 hours	48 hours	-----
VANC	Vancomycin, Random	80202	Refrigerate	Red Top (1)	8 hours	48 hours	-----
VANP	Vancomycin, Peak	80202	Refrigerate	Red Top (1)	8 hours	48 hours	-----
VANT	Vancomycin, Trough	80202	Refrigerate	Red Top (1)	8 hours	48 hours	-----
VB12	Vitamin B12	82607	Refrigerate	SST (1)	8 hours	24 hours	UNDETERMINED
VD25	Vitamin D-25 Hyd	82306	Refrigerate	SST (1)	Immediately spin and separate. Stable for 72 hours once separated.	7 days	1 year
WCUL	Culture, Wound	87070	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
WCULMRSA	Wound Culture R/O MRSA	87070	Refrigerate	Sterile Container (1)	24 hours	48 hours	-----
WET	Wet Mount	87210	Refrigerate	Swab (1)	24 hours	-----	-----
WGTT3	WHA Glucose Tolerance Test 3 Hour	82947,82950,82950,82952	Refrigerate	SST (1)	8 hours	48 hours	-----